SILC CUP

EN

This obstetrical instrument is utilized to assist a clinician in the delivery of an infant during childbirth.

Important note

Proper surgical procedures and techniques are the responsibility of the medical practitioner. Each physician must evaluate the appropriateness of the procedure based on his or her own medical training and experience.

General information

The Silc Cup is made of medical grade silicone and is available in two sizes, 50 and 60mmh inner diameter. The instrument is made in one piece and consists of a handle and a suction cup to be used with a pump and tubing. The cup has a smooth external surface, but there is a small longitudinal ridge, which makes it possible to observe any rotation. The inside of the cup is lined with small projections. The projections enable the air between the cup and the child’s head to be evacuated and ensure that the cup sticks firmly against the child’s head. The soft suction cup deforms, thereby ensuring optimal adhesion and minimizing trauma to the fetal scalp.

Pump requirements

- This obstetrical instrument has been developed for use in conjunction with an external vacuum source capable of precise vacuum regulation to ensure proper vacuum levels (up to -80 kPa/-600mmHg).
- The vacuum source should be designed for medical purposes and may be either electric or manual.
- We recommend using the Medela Basic 30 suction pump.
- Note: Wall suction is not to be used.**

Tubing requirements

The tubing connecting the instrument to the vacuum source is vital for safe and proper function of the instrument. To ensure this only Medela original tubing should be used.

Cleaning/Disinfection / before first use

- After use, soak the suction cups and tubings at room temperature for 30 minutes in a disinfection/cleaning solution A).
- After the reaction time, rinse the residue of the solution from the parts with water. Rinse the tubing.
- Clean suction cup and tubings separately with cleaning solution B).
- Eventually use a brush to remove residue.
- Rinse the tubing for at least 2 minutes with the cleaning solution B).
- The parts should be free from residue and foreign particles.
- Rinse the parts and the tubing with water (preferable fully desalinated) for 2 minutes.
- Thoroughly clean with sterile, non-bleach cloth.
- Hang the tubing up overnight to dry.
- A) Fluid aldehyde-free disinfection solution with cleaning effect for use as a soaking bath, bactericide, fungicide, limited virucide, with good material compatibility for stainless steel, non-ferrous material and plastics including silicone, slightly alkalik.
- For the validation of the procedure a 2% neodisher® Septo MED-Lösung was used.
- B) Fluid, pH-neutral, enzymatic cleaning solution for the treatment of instruments by machine or manually with very good material compatibility for stainless steel, non-ferrous material and plastics including silicone.

For the validation of the procedure a 2% neodisher® Septo MED-Lösung was used.

- A) Fluid, pH-neutral, enzymatic cleaning solution for the treatment of instruments by machine or manually with very good material compatibility for stainless steel, non-ferrous material and plastics including silicone.
- For the validation of the procedure a 1% neodisher® MediZym-Lösung was used.
- Disinfection of instruments mentioned:
Chemische Fabrik Dr. Weigert GmbH&Co.KG, Mühlenhagen 85, 20539 Hamburg, Germany (www.drweigert.de).
- The regulations of the manufacturer of the cleaning agent, resp. use and dangers must be kept.
- The use of other cleaning and disinfecting agents should be cleared with the hospital internal hygienic administrator.

- Sterilisation (also before first use)**
- Suction cups and tubings must be assembled and packed before sterilisation.
- Sterilization
- In the autoclave with saturated steam at 134°C, with triple pre-evaucation, sterilisation time 6 minutes.

Expected product life

- 60 cleaning/sterilisation cycles.

Safety rules

- If damages – especially tears - are determined, the parts may no longer be used.
- If the Silc Cup looks discoloured it should be discarded since this may indicate material deterioration.

- No modification of this equipment is allowed.
- Do not use it,
 - If the device is damaged,
 - If the device shows clear safety defects.
- Wall suction is not to be used.
- Vacuum extraction must only be performed or supervised by trained and experienced clinicians.
- The neonatal care staff must be aware of the indications, contraindications and precautions for assisted delivery and must watch for the signs of complications of vacuum assisted delivery.
- Operative vaginal delivery should not be performed when the probability of success is very low.
- Ensure against intrusion of extraneous tissues and ensure proper placement of cup.
- Attention: Never apply the cup to any portion of the infant’s face as this can cause tissue damage.
- Insertion should be performed carefully to avoid malfunctions, maternal or fetal trauma and maternal discomfort.
- The head, not just the scalp, must descend with each pull.
- Pull only during a contraction.
- Always apply traction at an angle allowing the stem of the cup to remain in-line or on-axis.
- Never apply rocking movements or torque to the device; only steady traction should be used.
- Discontinue vacuum extraction if there is evidence of significant scalp trauma or damage.
- The head must be delivered, or almost completely delivered, with no more than three pulls.
- The head must be completely delivered within 15–20 minutes of first applying the cup.
- Never exceed 10 minutes of cumulative traction time.
- Never exceed maximum recommended vacuum -80 kPa (-600 mmHg) or “pop offs” (2) – scalp trauma can occur.
- Discontinue vacuum extraction if the vacuum cup becomes disengaged or pops off twice.
- Do not pull up three times, if the second pull fails to produce good descent.
- Parents should be instructed to check the baby’s health.

Disposal

Dispose of according to local regulations.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

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